

Care & Independence

Sling Model:

Serial Number:

User Name:

*The sling model and serial number
are located on the label on the reverse of the sling.*

Care & Independence Ltd is a UK manufacturer registered with MHRA as a Class 1 Medical Device Manufacturer and this sling is a Class 1 medical device as defined in Article 51, Annex VIII Rule 1 of the Medical Device Regulation 2017/745.

COMFORT VALUE™ slings are designed and manufactured to meet or exceed BS EN 10535:2006 for the transfer of disabled persons. COMFORT VALUE™ slings are designed and manufactured by Care & Independence, a certified ISO 9001:2015 organisation.

Declaration


Care & Independence Ltd, declares under its sole responsibility that the aforementioned products to which this declaration relates, and which bear the CE marking, are in conformity with the applicable requirements of EU regulation 2017/745 as transposed in the laws of the Member States.

THIS USER GUIDE IS APPLICABLE
FOR THE FOLLOWING
COMFORT VALUE™ TOILETING
SLINGS:

CVT: COMFORT VALUE™ Toileting

CVPT: COMFORT VALUE™ Toileting Poly

ISSUE DATE: 02/12/21
VERSION: UG/CVTOILET-EN04
STATUS: APPROVED
AUTHORISED BY: Technical Director

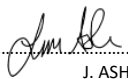
.....
J. ASH

1. GENERAL



- 1.1. WARRANTY:** The COMFORT VALUE™ slings covered in this User Guide have been awarded a THREE YEAR WARRANTY. Any manufacturing defect will be rectified free of charge. Product damage through wear and tear, neglect or accidental damage is not covered by the warranty.
- 1.2. COMPATIBILITY:** Care & Independence slings have been manufactured to work with most of the major hoist manufacturers' equipment. A compatibility statement is available on request.

2. SLING SAFETY



- 2.1 GENERAL:**
- 2.1.1 Only competent persons should ever use hoists and slings.
- 2.1.2 A risk assessment must be completed before using the sling, even if the hoist and sling is provided from the same manufacturer.
- 2.1.3 It is essential that the sling is fit for purpose; the correct size and type of sling for an individual is essential to ensure the safety and comfort of the person being transferred.
- 2.1.4 Product care and inspection instructions should be followed at all times.
- 2.1.5 **No sling must ever be used to lift more than its rated load** (see label).
- 2.1.6 The intended sling and hoist combination should be compatible.
- 2.1.7 **If there is any doubt about sling safety, remove the sling from use until checked and authorised for use by a competent person.**

2.2 SLING INSPECTION:

- 2.2.1 Care & Independence recommend pre-use sling checks by competent person.
- 2.2.2 Slings are Class One Medical Devices and are subject to Lifting Operations and Lifting Equipment Regulations (LOLER) Examinations. A LOLER examination may only be conducted by a person trained and qualified to do so. Space is provided on the sling label to record LOLER details.



3. DAMAGED OR FAULTY SLINGS



Care & Independence have a strict *Returns & Repair Procedure* to ensure the safety of all involved.

Please **do not** send slings and bathing products back without completing the Return/Repair form.

4. PRODUCT CARE



4.1 GENERAL CARE:

- 4.1.1 Repeated washing may lead to sling deterioration. **Care & Independence strongly recommend replacing the sling at the first signs of wear or damage, as opposed to after a specified time frame.**
- 4.1.2 Each sling is marked with its maximum wash temperature.
- 4.1.3 Discolouration to some materials or fading labels may occur as a result of laundering. **NOTE: It is a health & safety requirement that the sling label must be legible throughout the sling's working lifetime.**

4.2 WASHING THE SLING: Every sling is marked with its maximum wash temperature. Close hook & loop fastenings before laundering to prevent damage to the fabric and webbing.

4.3 DECONTAMINATION:

Slings can withstand a strong decontamination routine but regular use may impair the lifespan of the sling.

A copy of these instructions should be provided to the person(s) laundering the sling.



- Wash gently at 40°C. Use a mild soap solution or a non-biological detergent without bleaching additives.
- Do not add bleach or stain removers.
- Air dry, cabinet dry or tumble dry on a cool cycle (below 60°C). Do not tumble dry on a hot setting.
- Do not place on hot pipes or heaters. Keep away from flames.
- Do not iron.


FURTHER INFORMATION: Available to download from our website



- Sling Care Instruction Guide (includes decontamination advice).
- Returns & Repair Procedure

5. GETTING TO KNOW YOUR SLING

5.1 THE ACCESS-STYLE SLING

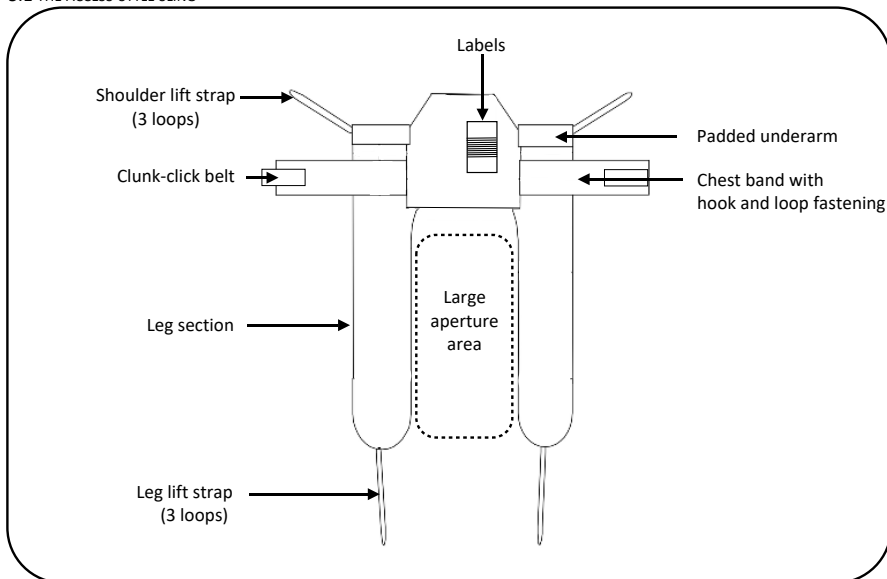

5.2 SLING LABELS EXPLAINED

There are four labels associated with each sling.

Model Size Order code Year of Manufacture	
RECOMMENDED WASHING 	PRE-USE CHECKS
DISINFECTION GUIDANCE	MULTI-USER DISCLAIMER
NAME: _____	

5.2.1 SLING INFORMATION

Printed label sewn onto back of sling. Provides useful information such as Sling Model Type, Washing instructions, Pre-Use Checks and a space to write User's name. A QR code is also featured that once scanned leads direct to product webpage where support resources are available.

LOLER Thorough Examination Record		
TO DATE	EXAMINER NAME	NEXT TO DATE DATE

5.2.2 LOLER RECORD

Printed label sewn onto back of sling. Convenient to record the sling's LOLER checks



5.2.4 QUALITY TAG (Disposal)

Attached to the side of the sling. Shows factory quality control approvals. Reverse side is Union flag.




5.2.3 PRODUCT INFORMATION

Woven label sewn onto back of sling. Contains serial number, Health & Safety Regulation Compliance mark, Maximum Load Capacity and QR code.

6. FITTING THE SLING

Before you start, check that the sling is in safe and clean condition and is the appropriate size and model for the person to be transferred. There are a variety of ways to use and adjust this sling, so please read carefully.

6.1 IN A SEATED POSITION

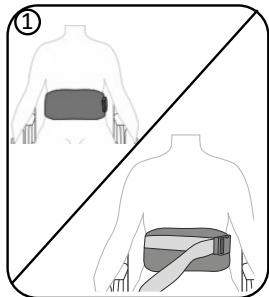


6.1.1

The waist support should be fitted as low as possible around the torso with arms outside the sling.


- Fasten the hook and loop band.
- Secure the clunk-click belt.

NOTE: The hook and loop band should always be closed when not in use.

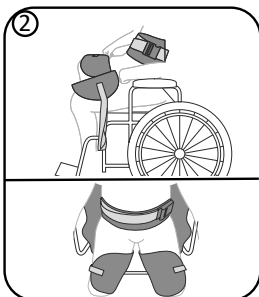

6.1.2

Bring the leg sections around the hips and feed them under and up between the thighs.

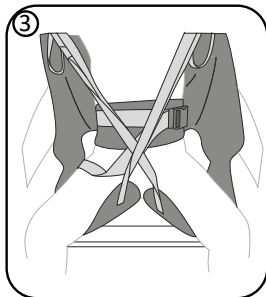


6.1.3

Following a risk assessment, leg straps can be crossed (RECOMMENDED). Alternatively continue without crossing.

- Cross the leg straps over one another by threading one strap through the other by passing one strap through the section beneath the bottom loop.
- Attach leg straps to the hoist first then shoulder straps.
- Check for snagging.
- As the service user begins to be lifted and the body thins out re-tighten the clunk-click belt.
- Complete the transfer.

6.2 POSITIONING A USER BACK INTO A CHAIR

To assist guidance into the chair, a preferred technique is to use your body weight from the front, pushing on the bottom of the leg lift straps, as the user lowers into the chair.

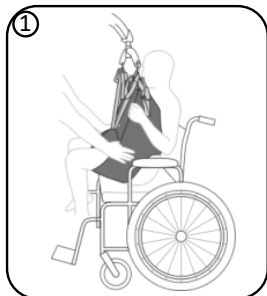

Available to download from our website

- Sling Pre-Check Inspection Guide
- Loop Attachment Guide
- Sling Sizing Guide

7. ADDITIONAL SUPPORT AVAILABLE


SCAN SLING LABEL QR CODE FOR;

- Product information
- User Guide
- Fitting video
- Sling Care Instruction Guide
- Sling Pre-Check Inspection Guide
- Sling Sizing Guide



Free product training is also offered. Please contact us for further information.



- Expert advice
- Full support



- Faded label replacement
- Sling repair
- LOLER examinations

This part has been intentionally blank for notes

Comfort VALUE™ USER GUIDE

Care & Independence

www.careandindependence.com

Care & Independence Ltd,
Unit 3, Telford Business Centre,
Telford Road, Bicester, OX26 4LD

033 012 80808
sales@careandindependence.com

Monday - Friday 08.30 -17.00

*If a third party supplier has provided the GLOVE™ sling,
you may like to write their contact details here:*



©2021 Care & Independence Ltd All Rights Reserved.
Contents of this publication are intended as a guide only. Although due care and attention has been given to ensure that instructions are accurate and correct, in case of doubt, please refer to your prescribing clinician or contact the Care & Independence team on 033 012 80808. Care & Independence Ltd reserve the right to modify, add or delete details and is under no obligation to notify the user of amendment. All data is correct at time of printing.