

Just wipe & go... with Hygenex Disinfectant Wipes




- ✓ **One-step process**
- ✓ **Faster disinfection**
- ✓ **Great flexibility**
- ✗ **No carrying bulky bottles**
- ✗ **No aerosolisation waste**
- ✗ **No inhalation risk**

Sanitise equipment and surfaces to prevent infection – with no need to clean with detergent first.

Hygenex Disinfectant Wipes can be used to...



- **Clean and disinfect patient-shared equipment**
- **Clean non-invasive medical devices before sterilisation**
- **Sanitise surfaces where risk of infection is high**

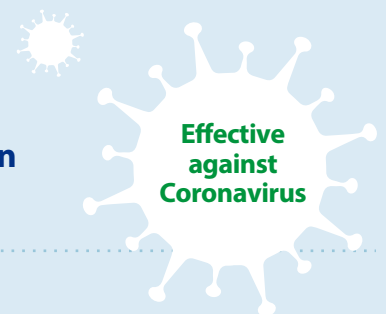

How to use:



- Step 1 -

Assess risk

Use PPE as agreed in your risk assessment guidelines.



- Step 2 -

'S- shape' technique

Remove wipe from dispenser and use a 'S-shape' technique, keeping the wipe flat (not scrunched). Move the wipe from clean to dirty. Avoid re-visiting the same place with the same wipe.



- Step 3 -

Fresh wipe

Once the wipe becomes soiled or dry, get a new wipe and continue.



- Step 4 -

Dispose

Dispose of used wipes in the appropriate bin.



- Step 5 -

Dry

Allow the surface to dry before use.