


## JoernsSling Weasuring SurgerOxfordOxford Deluxe Transport

CAUTION: This document is intended as a guide only and in no way replaces an individual patient risk assessment. Patient specific conditions, functional independence measure (FIM) and weight distribution may influence the most suitable sling size as well as the most suitable type.

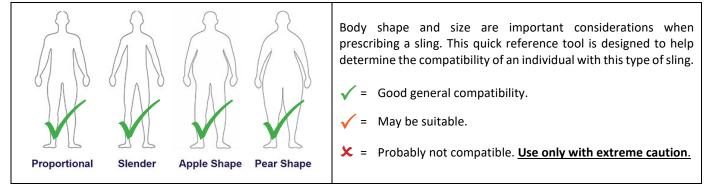



## Key Dimension: High Waist Circumference

Take the client measurement as shown in the diagram below and reference it against the corresponding measurements in the colour-coded chart to help determine the most suitable sling size.

	Sling Size	Dimensional Range		Oxford slings have colour-coded
		Min	Max	binding matching this chart to help quickly identify the size.
	XS	550	670	
000	S	650	810	
	м	790	1040	
	L	1020	1290	
(	XL	1270	1510	

For further assistance, core dimensions for each sling size are provided on the next page.

Selecting a sling for an individual involves more than simply measuring their size. Other considerations, as part of a comprehensive risk assessment, should include, but are not limited to:

- Type of transfer (e.g. bed to chair, chair to toilet, etc)
- Level of support required
- Patient conditions
- Physical and cognitive ability of the patient
- Abilities and limitations of the carer

- Compatibility of the sling with other equipment
- Is a commode aperture required?
- Is the sling to be used for bathing?
- Potential obstacles and hazards during transfer


It is critical to assess all possible risks when planning to lift and transfer a patient from one surface to another. Failure to assess the risk correctly may result in serious injury to the patient and/or caregiver.



For further information on sling use, please contact your local Oxford Sales Specialist on +44 (0)344 811 1158.

## Core Sling Dimensions: Oxford Deluxe Transport

Deluxe Transport						
Sling Size	Dimension A	Dimension B	Dimension C			
xs	-	-	-			
S	670mm	290mm	860mm			
м	840mm	340mm	1010mm			
L	1100mm	400mm	1060mm			
XL	-	-	-			

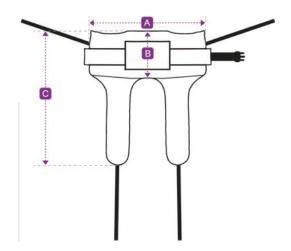

\*All dimensions are subject to manufacturing tolerances.

## **Custom Sling Service:**

Joerns Healthcare offer a custom sling service should you have any non-standard or patient specific requirements. Please contact Joerns Healthcare for more information and guidance. Extended lead times apply.

