





SIDE FIRE



Sample right first time

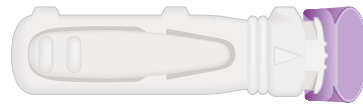


How to use Unistik® 3

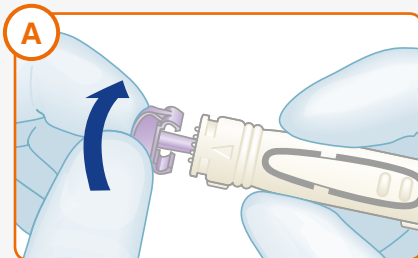


Hold Unistik® 3 from side. **TWIST** cap a quarter turn to unlock. You'll feel the cap advance forward. **DO NOT PULL!**

Continue twisting until you feel it separate from the device. Then pull the cap.

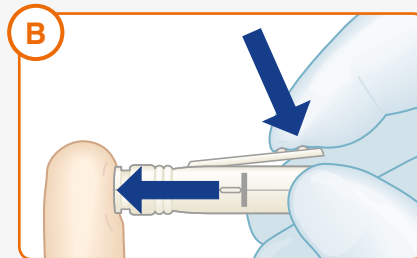

Press Unistik® 3 against finger and **PRESS** the release button.

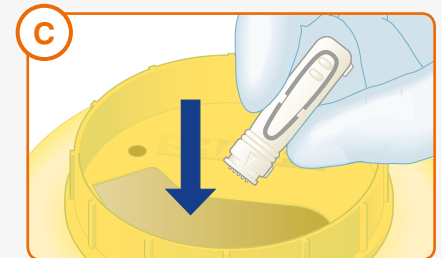

DISPOSE of the used Unistik® 3 in a suitable sharps container.

Do not use if cap has been previously removed. Single use for safety.



Available in five needle gauge and depth specifications. Enabling you to choose the right lancet for every test.

	Unistik 3 Gentle 30G 1.5mm	Frequent blood glucose testing and paediatrics	Box of 50: AT1025 Box of 100: AT1022 Box of 200: AT1024
	Unistik 3 Comfort 28G 1.8mm	Frequent blood glucose testing and other tests e.g. cholesterol and haemoglobin	Box of 50: AT1045 Box of 100: AT1042 Box of 200: AT1044
	Unistik 3 Normal 23G 1.8mm	Blood glucose testing and tests requiring larger samples e.g. INR, liver function tests, cholesterol, and haemoglobin	Box of 50: AT1008 Box of 100: AT1002 Box of 200: AT1004
	Unistik 3 Extra 21G 2.0mm	Tests requiring larger samples e.g. cardiac markers, coagulation, and blood gases	Box of 100: AT1012 Box of 200: AT1014
	Unistik 3 Neonatal & Laboratory 18G 1.8mm	Neonatal Heelstick samples and tests requiring very large samples e.g. chemistries electrolytes, and haematology	Box of 100: AT1052 Box of 200: AT1054

The appropriate device to use will depend on the blood volume required, which will vary depending on the analyser type and brand.

© Copyright Owen Mumford 2018
U3-HTU-UK-POSTER/MCC/0818/01

Unistik® 3